























VACUETTE® Urine Tubes (sterile)

456007	Z Urine No Additive, round base	6.5ml	13x100	1200pcs. (50pcs./rack)	
456065	Z Urine No Additive, round base, non-ridged	6.5ml	13x100	1200pcs. (50pcs./rack)	
455028	Z Urine No Additive, conical base, non-ridged	9.5ml	16x100	1200pcs. (50pcs./rack)	
455007	Z Urine No Additive, round base, non-ridged	10.5ml	16x100	1200pcs. (50pcs./rack)	
454239	Urine Culture (boric acid), round base, non-ridged	4ml	13x75	1200pcs. (50pcs./rack)	
456009	Urine Culture (boric acid), round base	6.5ml	13x100	1200pcs. (50pcs./rack)	
455029	Urine Culture (boric acid), conical base, non-ridged	9.5ml	16x100	1200pcs. (50pcs./rack)	
455003	Urine Culture (boric acid), round base, non-ridged	10.5ml	16x100	1200pcs. (50pcs./rack)	
455048	Urine Stabilur, round base, non-ridged	10,5ml	16x100	1200pcs. (50pcs./rack)	

Urine Sets

453060	Urine Set 6.5ml 456007 Urine Tube (sterile) + 450251 Urine Transfer Device 200 pcs. Urine Tubes + 200 pcs. Urine Transfer Devices, 4 racks of 50 + 4 bags of 50	
453070	Urine Set 9.5ml 455028 Urine Tube (sterile) + 450251 Urine Transfer Device 300 packs of 1pc. Urine Tube + 1 pc. Urine Transfer Device	
453050	Urine Set 10.5ml 455007 Urine Tube (sterile) + 450251 Urine Transfer Device 200 pcs. Urine Tubes + 200 pcs. Urine Transfer Devices, 4 racks of 50 + 4 bags of 50	
453022	Urine Culture Set 4ml 454239 Urine Culture Tube (sterile) + 450251 Urine Transfer Device 300 packs of 1 pc. Urine Culture Tube + 1 pc. Urine Transfer Device	
453041	Urine Culture Set 6.5ml 456009 Urine Culture Tube (sterile) + 450251 Urine Transfer Device 300 packs of 1 pc. Urine Culture Tube + 1 pc. Urine Transfer Device	
453080	Urine Culture Set 9.5ml 455029 Urine Culture Tube (sterile) + 450251 Urine Transfer Device 300 packs of 1 pc. Urine Culture Tube + 1 pc. Urine Transfer Device	
453085	Urine Culture Set 10.5ml 455003 Urine Culture Tube (sterile) + 450251 Urine Transfer Device 300 packs of 1 pc. Urine Culture Tube + 1 pc. Urine Transfer Device	

VACUETTE® Urine Accessories

450251	Urine Transfer Device, 10cm straw especially for use with Urine Beaker	600 pcs. (12 bags of 50 pcs.)	
450252	Urine Transfer Device, 20cm straw especially for use with 24hr Urine Collection Container	600 pcs.	
724310	Urine Beaker with Intragrated Transfer Device	200 pcs., single-packed	
724320	Urine Beaker, yellow lid, sterile	300 pcs., single-packed	
724300	Urine Beaker with Safety Stopper	100 pcs., single-packed	
724315	24hr Urine Collection Container white lid, amber container	34 pcs.	

Worldwide Addresses

<p>Austria / Europe Bad Haller Strasse 32 A - 4550 Kremsmünster Phone: (+43) 7583 6791 - 0 Fax: (+43) 7583 6318 office@at.gbo.com</p>	<p>Switzerland St. Leonhardstrasse 31 CH-9000 St. Gallen Phone: (+41) 71 228 55 22 Fax: (+41) 71 228 55 21 office@ch.gbo.com</p>
<p>Germany Krablerstrasse 127 D - 45326 Essen Phone: (+49) 201 861 86 - 0 Fax: (+49) 201 861 86 - 12 office@de.gbo.com</p>	<p>Spain Parque Empresarial Inbisa 1 Av. Camino de lo Cortao 30, Nave 12 San Sebastian de los Reyes 28700 Madrid Phone: (+34) 91 652 77 07 Fax: (+34) 91 652 33 35 info@vacuette.es</p>
<p>United Kingdom Stroudwater Business Park Brunel Way Stonehouse, Glos. GL10 3SX Phone: (+44) 14 53 82 52 55 Fax: (+44) 14 53 82 72 77 office@uk.gbo.com</p>	<p>Brazil Av. Affonso Pansan N° 1.967 13.473-620 Vila Bertini Americana - SP Phone: (+55) 19 3468 9600 Fax : (+55) 19 3468 9621 office@br.gbo.com</p>
<p>The Netherlands Albert Einsteinweg 16 2408 AR Alphen a/d Rijn Phone: (+31) 172 42 09 00 Fax: (+31) 172 44 38 01 office@nl.gbo.com</p>	<p>Hungary Fertősor 7 H - 9200 Mosonmagyaróvár Phone: (+36) 96 213 088 Fax: (+36) 96 213 198 office@hu.gbo.com</p>
<p>U.S.A 4238 Capital Drive Monroe, North Carolina 28112 Phone: (+1) 704 261 7800 Fax: (+1) 704 261 7899 Customer service: 888-286-3883 office@us.gbo.com</p>	<p>France 3 à 7 Rue du Cap Horn BP31 Les Ulis 91941 Courtaboeuf Phone: (+33) 169 86 25 25 Fax: (+33) 169 86 25 35 office@fr.gbo.com</p>
<p>Egypt Dr. Hisham Ouda Regional Manager Middle East 15 Ahmed Kamal Street Heliopolis Cairo - Egypt Phone: + 202 62 18 706 / + 20 123 217 400 Fax: + 202 62 18 708 Hot Line: + 20 12 21 89 180 hisham.ouda@gbo.com</p>	<p>Portugal R/C Bloco A Areia - Arvore Rua de Estrada Velha, 1281 Phone: (+351) 252 647 721 Fax: (+351) 252 647 722 info@vacuette.pt</p>

www.gbo.com

980005 Rev. 07.2007 e



> One Step Ahead

VACUETTE® Urine System

► Improved hygiene

Improved safety for laboratory and hospital personnel through the use of a closed system; contamination is virtually impossible when decanting into a secondary vessel.

► Safe and convenient transport

VACUETTE® Urine Tubes are hermetically sealed, sterile, unbreakable and space saving.

► Pre-analytical standardisation

Errors produced as a result of collection, transportation and storage are reduced to a minimum. Sterile VACUETTE® Urine Tubes have an exact dosage vacuum and are guaranteed to meet highest quality standards.

► Highly efficient and cost saving

Collection, transportation, centrifugation, strip tests, analysis and storage can all be executed in a single sterile tube.

Recommendations to guarantee a stable sample quality:

- ⊕ In cases where the sample remains in the Urine Beaker for a longer period of time, the Urine Transfer Device should be used to thoroughly mix the sample, thus distributing the sedimentation throughout the sample prior to decanting.
- ⊕ Exclusive use of a sterile Urine Beaker for urine collection will delay bacterial growth, which in turn could influence sample quality.
- ⊕ When using VACUETTE® Urine Culture Tubes, tubes should be inverted several times so as to mix the urine and preservative homogeneously.
- ⊕ Recommended centrifugal conditions for conical base tubes: 5 Min. / 400 g.



The practical solution for standardised pre-analysis

Instructions for use:

- Urine Beaker**
Fill the sterile Urine Beaker with a urine sample and close the lid of the beaker. Push the Urine Transfer Device through the Safety Stopper until the straw is immersed in the urine sample.

Urine Beaker with Integrated Transfer Device
Fill the sterile Urine Beaker with a urine sample and close the lid of the beaker.
- Urine Beaker**
Place the VACUETTE® Urine Tube in the Urine Transfer Device until the needle penetrates the Safety Stopper. The urine will then automatically flow into the Urine Tube in accordance with the exact defined vacuum.

Urine Beaker with Integrated Transfer Device
Remove the protective sticker from the lid. Insert the Urine Tube into the Integrated Urine Transfer Device until the Safety Stopper is fully penetrated. The urine will then flow into the Urine Tube in accordance with the exact defined vacuum.
- Urine Beaker**
Remove the filled Urine Tube from the Urine Transfer Device.

Urine Beaker with Integrated Transfer Device
Remove the filled Urine Tube from the Integrated Urine Transfer Device in the lid of the Urine Beaker.
- Urine Beaker**
Invert the Urine Tube several times after filling. Dispose of the Urine Transfer Device and the Urine Beaker accordingly.

Urine Beaker with Integrated Transfer Device
Replace the protective sticker on the lid, thus preventing needlestick injuries during disposal. Invert the Urine Tube several times. Dispose of the Urine Beaker with integrated Urine Transfer Device accordingly.




24 HR URINE COLLECTION CONTAINER
the practical solution for urine collection over a 24 hour period.



URINE BEAKER
with integrated urine transfer device; Sterile, break-proof and leak-proof.



URINE TRANSFER DEVICE
for the convenient sample transfer from the collection vessel.

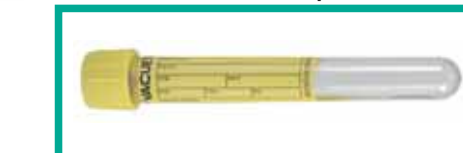

VACUETTE® URINE TUBE
round base (sterile).



VACUETTE® URINE TUBE
conical base (sterile).