



MiniCollect® Safety Lancets

Gentle & Caring for Patients Both Young and Old



- Safe and hygienic capillary blood collection
- More precision for simple sample collection
- Unique identification of different types with colour code
- Practical handling for increased collection comfort

More Safety



- Absolute safety due to the automatic retraction of the blade into the plastic housing after puncture
- The blade is only activated when the lancet has been positioned on the skin and the activation button is pressed
- Designed for single-use only

Precision and Simple Handling




- Each version ensures a consistent puncture depth for straightforward sample collection
- The standardised blade positioning means reliably consistent puncture alignment
- Ergonomic design for more handling comfort
- Self-explanatory application saves time
- Small, compact product design saves space in disposal container

Clear Identification

Available in three different puncture depths, which are identifiable with three different colours.

Application

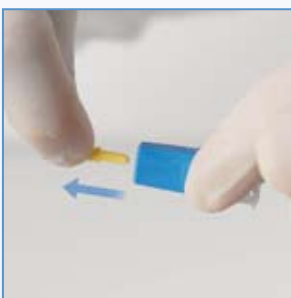

1 Remove the protective cover



2 Position the lancet



3 Puncture the skin by pressing button



4 Discard first drop of blood

Item Numbers

MiniCollect® Safety Lancets (available at the end of 2008)

Item n°.	Description	Blood Volume	Inner pack	Outer pack
450427	MiniCollect® Safety Lancet, pink, penetration depth 1 mm	Low blood volume	200 pcs.	2000 pcs.
450428	MiniCollect® Safety Lancet, green, penetration depth 1,5 mm	Medium blood volume	200 pcs.	2000 pcs.
450429	MiniCollect® Safety Lancet, blue, penetration depth 2 mm	High blood volume	200 pcs.	2000 pcs.