# VACUETTE®

## VACUETTE® Holdex®

### VACUETTE® Holdex®

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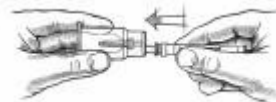
### VACUETTE® Holdex®

The **VACUETTE®** Holdex® holder is a sterile single-use disposable holder with an off-center female luer at the top of the holder and epoxy bonded stainless steel needle on the underside of the holder, recommended for use with commercially available butterfly and luer needles in routine venipuncture procedures. This device is to be used by properly trained healthcare professionals only in accordance with these instructions

#### GENERAL INSTRUCTIONS FOR USE:

1. Grasp the Holdex® holder in one hand and place a butterfly needle or luer needle onto the female luer located at the top of the holder. Make sure the needle is securely seated.

2. Remove the shield from the luer needle. The system is now ready for the venipuncture procedure.

3. After venipuncture, dispose of Holdex® and needle in approved disposal bin.

#### WARNINGS:

Do not reshield needle. Dispose of used needle in appropriate "sharps" bin in accordance with the policies and procedures of your facility. Keep hands behind needle at all times during use and disposal.

This Product Contains Dry Natural Rubber

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greiner bio-one GmbH - Bad Haller Straße 32 - A-4550 Kremsmünster - Austria